
BE1221 Vibio Bed Shaker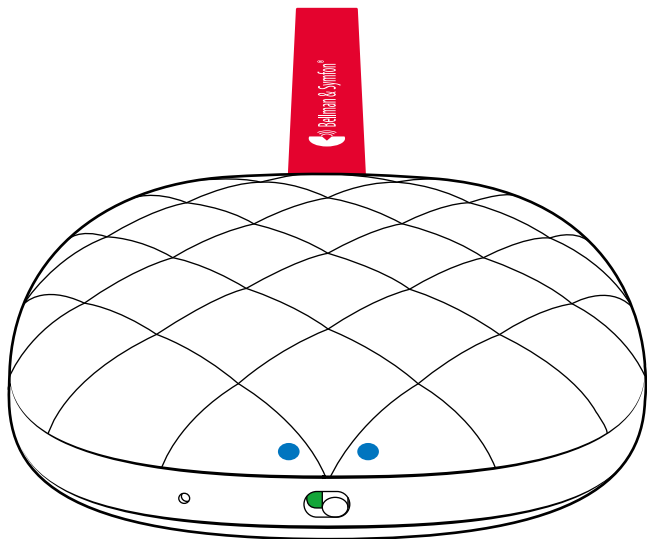

BE1221 Vibio Bed Shaker

-
- 4 User manual

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EN

ES

FR

CN_{SI}

CN_{TR}

JP

Read this first

This booklet guides you on how to use and maintain your new Vibio Bed Shaker. Ensure you read this booklet carefully, including the **Warnings** section. If you have additional questions, contact your hearing care professional.

About BE1221 Vibio Bed Shaker

Intended user group

The intended user is a person with a wish to receive augmented signals.

Intended user

The intended user group consists of people of all ages.

Principle of operation

The Vibio Bed Shaker is typically placed under the pillow or mattress and the alarm is set in the Vibio mobile phone application. When the alarm goes off, the bed shaker vibrates under the pillow or mattress to awake the user. Once the alarm is set, the bed shaker works independently.

The device will not restore normal hearing and will not prevent or improve a hearing impairment or deafness resulting from organic conditions.


Welcome!

What you'll need

- A mobile phone with iOS 15 / Android 8 or later.
- A USB power adapter

What's in the box


- 1 Charging cable
- 2 Vibio Bed Shaker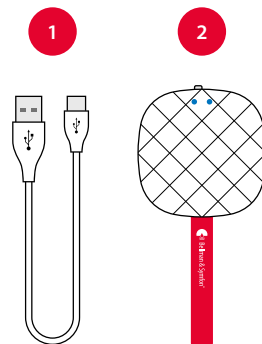

BE1221 Vibio Bed Shaker

Look for the blinking eyes

- White means it's activated by an alarm or communicating with your mobile.
- Blue means it's ready for pairing.
- Green means it's charging.
- Yellow means you have an incoming call or message.
- Red means the battery level is low.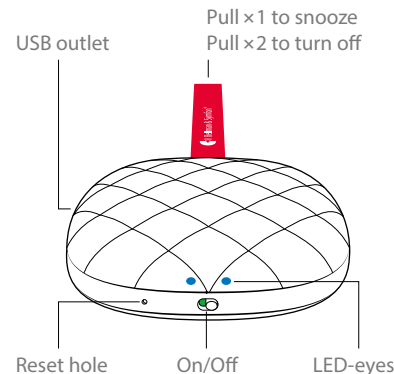


Installing Vibio

1 Power it up

Connect the charging cable to your Vibio and plug the connector into a USB power adapter or a USB computer outlet.

2 Turn it on

Slide the On/Off switch to the right and keep Vibio close to your mobile.

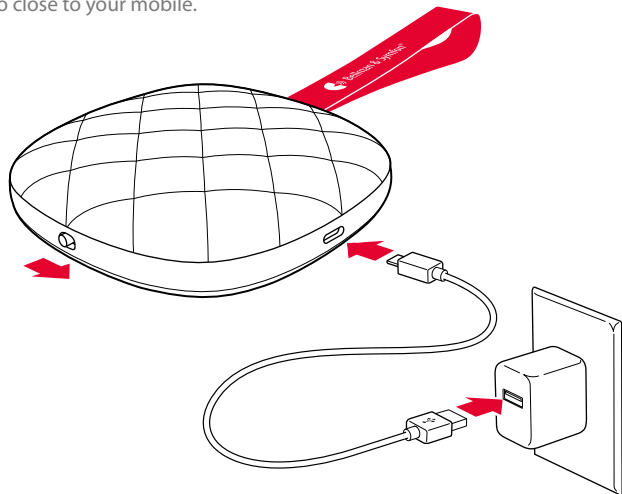3 Install the app

Download the **Vibio** app from the App Store® or Google Play™ and follow the setup instructions.

Using the app

Managing alarms


- Add.** Tap the plus button to add a new alarm.
- Edit.** Tap the pencil button to edit an alarm.
- Delete.** Tap the minus button to delete an alarm.
- On/Off.** Tap the ON or OFF button to enable or disable an alarm.

Monitoring Vibio

- Connection.** The Vibio icon shows the connection status.
- Battery.** The battery icon shows the current battery level.

Learning more

- Help.** Tap the Menu > Help to learn more about the app.

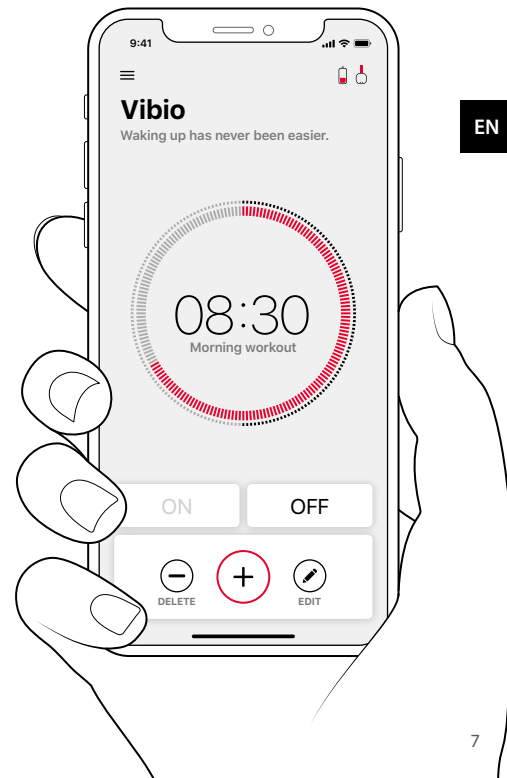Important safety information

This section contains important information about safety, handling and operation. Keep this leaflet for future use. If you are just installing the device, this leaflet must be given to the householder.

Hazard warnings

- Failure to follow these safety instructions could result in fire, electric shock, or other injury or damage to the device or other property.
- Keep this device out of reach of children under 3 years.
- Be aware that alarms and notifications can be missed if the batteries run out.
- Do not use or store this device near any heat sources such as naked flames, radiators, ovens or other devices that produce heat.
- Use only power adapters and battery types that are specified in this leaflet.
- Protect cables from any potential source of damage.
- Do not dismantle the device; there is a risk of electric shock. Tampering with or dismantling the device will void warranty.
- This device is designed for indoor use only. Do not expose the device to moisture.
- Protect the device from shocks during storage and transport.
- Only refer a damaged or faulty product to qualified service personnel.

Information on product safety

- Failure to follow these instructions could result in damage to the device and void the warranty.
- Do not expose batteries to fire or to direct sunlight.
- Do not drop your device. Dropping onto a hard surface can damage it.
- Do not use the device in areas where electronic equipment is prohibited.
- If you encounter problems with your device, contact the point of purchase, your local Bellman & Symfon office or the manufacturer. Visit bellman.com for contact information.
- Rechargeable batteries should be charged fully before using the device for the first time. Only charge batteries in a temperature between 0°– 35° C, 32°– 95° F.
- Rechargeable batteries should only be changed by Bellman & Symfon or an authorized service center. There is a risk of explosion if the batteries are not replaced correctly.
- This device should not be used on aircrafts unless specifically permitted by flight personnel.

Configuration options

This bed shaker can be used with the following mobile phone models and operating systems:

- iPhone 6S and later or Modern Android devices
- iOS15 or Android 8 and later.

Operating conditions

Operate the device in a dry environment within the temperature and humidity limits stated in this leaflet. If the product gets wet or is exposed to moisture, it should no longer be regarded as reliable and should therefore be replaced.

Cleaning

Disconnect all cables before you clean your device. Use a soft, lint-free cloth. Avoid getting moisture in openings. Do not use household cleaners, aerosol sprays, solvents, alcohol, ammonia, abrasives or compressed air.

Service and support


If the product appears to be damaged or doesn't function properly, follow the instructions in the user guide and this leaflet. If the product still doesn't function as intended, contact your local hearing care professional for information on service and warranty.

Warranty conditions

Bellman & Symfon guarantees this product (excluding the battery) for two (2) years from date of purchase against any defects that are due to faulty materials or workmanship. This guarantee only applies to normal conditions of use and service, and does not include damage resulting from accident, neglect, misuse, unauthorized dismantling, or contamination howsoever caused.

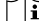This guarantee excludes incidental and consequential damage. Further the warranty does not cover Acts of God, such as fire, flood, hurricanes and tornadoes. This warranty gives you specific legal rights and you may also have other rights that vary from state to state. Some states or jurisdictions do not allow the limitation or exclusion of incidental or consequential damages, or limitations on how long an implied warranty last so the above limitation may not apply to you. This guarantee is in addition to your statutory rights as a consumer. The above warranty may not be altered except in writing signed by both parties hereto.

Symbol explanation

-  Indicates the manufacturer's serial number so that a specific device can be identified.
-  Indicates the device manufacturer.
-  This symbol indicates the need for the user to consult the instructions for use.
-  This symbol indicates that caution is necessary when operating the device or control close to where the symbol is placed, or that the current situation needs operator awareness or operator action in order to avoid undesirable consequences.
-  To identify a place where information may be found, especially in an emergency.
-  Indicates the temperature limits to which the device can be safely exposed.
-  Indicates the range of humidity to which the device can be safely exposed.

Troubleshooting

If	Try this
The LEDs are blinking in red	The battery level is low. Charge Vibio using the supplied cable.
The LEDs don't light up	The batteries are depleted. Charge Vibio for about 1.5 hour.
Vibio doesn't connect to my mobile	<p>Start by checking the Bluetooth connection on the app home screen. If the Vibio icon is red and crossed out, try the following:</p> <ol style="list-style-type: none">1 Move within Bluetooth range. Move your mobile closer to Vibio.2 Turn Bluetooth Off and On. Swipe to open the Control Center / Quick Settings Panel on your mobile and tap the Bluetooth icon. <p>If it still doesn't work, you need to reset and reconnect the Vibio Bed Shaker and app manually. Try the following:</p> <ol style="list-style-type: none">1 Force-quit the app. Note that it's not enough just to close it.2 Disconnect Vibio. Go to Settings > Bluetooth. Tap ⓘ / ⚙️ and Forget This Device to disconnect "Vibio". Make sure Bluetooth is on.3 Reset Vibio. Pull and hold the strap until the LEDs starts to flicker in blue. Release and pull it again until the LEDs starts to breathe.4 Setup Vibio. Start the Vibio app again and follow the instructions.

Website:

Technical specifications

Dimensions	27 × 94 × 94 mm, 1.1 × 3.7 × 3.7"
Weight	153 g, 5.4 oz.
Power and battery	Battery power: 1 × 3.7 V AA Lithium-ion Power consumption: < 1000 mA, Idle: < 100 µA Operating time: Up to 40 days / charge Charging time: Up to 1.5 hour @ 500mA Charging current: Up to 500 mA, 5V DC
Cables and connections	Cable length: 1 m, 39" Contacts: USB type A to USB Micro-B
Compatibility	iOS15 / Android 8 and later iPhone 6S and later / Modern Android devices
Frequency and coverage	<ul style="list-style-type: none">■ Bluetooth frequency: 2400 – 2483.5 MHz■ Output power: Max 2.5mW (4dBm)■ Coverage: Up to 30 m, 33 yd, free field.■ Connectivity: Bluetooth® 5
Temperature during transportation and storage	-10° to 50° C
Temperature during operation	0° to 35° C
Humidity during operation	15% to 95%, non-condensing
Atmospheric pressure during operation, transportation and storage	700 hpa to 1060 hpa
In the box	<ul style="list-style-type: none">■ BE1221 Vibio Bed Shaker■ USB charging cable

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FCC compliance statement

This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) this device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

Warning: Changes or modifications to this unit not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

Note: This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

IC compliance statement

This device contains licence-exempt transmitter(s)/receiver(s) that comply with Innovation, Science and Economic Development Canada's licence-exempt RSS(s). Operation is subject to the following two conditions: (1) This device may not cause interference.

(2) This device must accept any interference, including interference that may cause undesired operation of the device.

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FCC ID: 2APAKBE1221 **IC ID:** 6693A-BE1221

Bluetooth® certified

This device includes a Bluetooth module which has passed the BQB certificate. The Bluetooth word mark and logos are owned by the Bluetooth SIG, Inc., and any use of such marks by Bellman & Symfon is under licenses.

Model, type and classification

The information is available at the back of the product.

Product disposal and recycling information

According to local laws and regulations, your product should be disposed of separately from household waste. When this product reaches its end of life, take it to a collection point designated by local authorities. The recycling of your product will help conserve natural resources and ensure that it is recycled in a manner that protects human health and the environment.

Battery disposal information

The product contains batteries that according to local laws and regulations must be disposed of separately from household waste. Take used batteries to a collection point designated by the authorities.

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Revision: BE1221_151MAN3.0

Date of issue: 2025-01-08

TM and © 2025 Bellman & Symfon AB. All rights reserved.

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Bellman & Symfon North America Inc.
Bellman & Symfon Asia Ltd.
Bellman & Symfon Group AB